Space[®]plus

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The screenshot shows the login screen for the Service Tool Plus Generation. On the left, there is a green header with the text "Service Tool Plus Generation" and the version number "2.2.2". Below this is the B. Braun logo with the tagline "SHARING EXPERTISE". The main area is titled "Willkommen Service-Admin" and contains a list of user roles: "Service-Admin" (selected), "Servicetechniker*in", "Konfigurationsnutzer*in", "B. Braun Admin", and "B. Braun Produktion". To the right of the list is a password field labeled "Passwort" and two buttons: "Anmeldung" and "Beenden".

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The screenshot shows the configuration interface for the Service Tool Plus Generation. The 'Allgemeines' tab is selected. The 'Verbindungseinstellungen' section is expanded, showing options for serial and network connections. The 'Produktfamilie' and 'Gerätetyp' fields are highlighted with a yellow box. The 'Produktfamilie' field has 'SpacePlus' selected, and the 'Gerätetyp' field has 'Pumpe' selected. The 'OK' and 'Abbrechen' buttons are visible at the bottom right.

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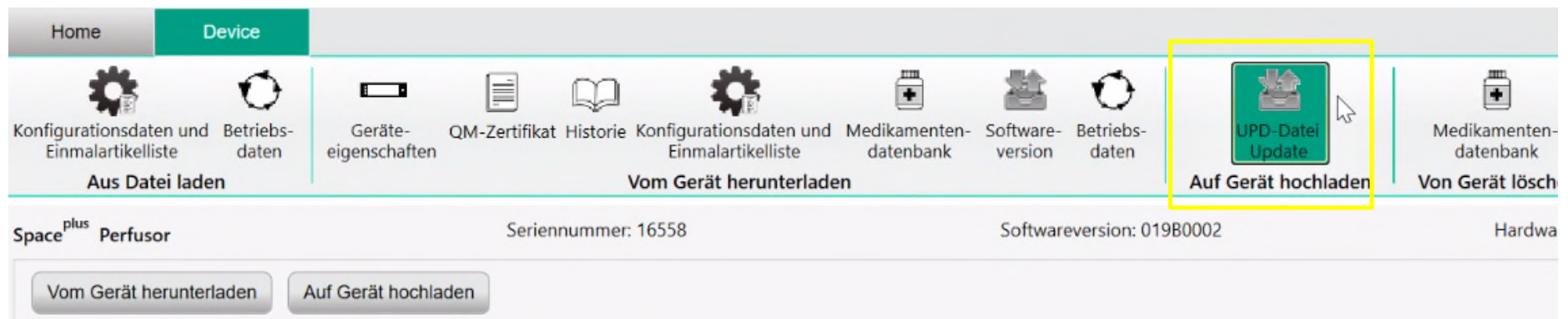
The screenshot displays the 'Device' tab of the Space plus Perfusor software interface. The top navigation bar includes 'Home' and 'Device'. Below this, a horizontal menu contains several icons and labels: 'Aus Datei laden' (with a gear icon), 'Geräte-eigenschaften' (with a device icon), 'QM-Zertifikat' (with a document icon), 'Historie' (with a book icon), 'Vom Gerät herunterladen' (with a gear icon), 'Medikamentendatenbank' (with a pill icon), 'Softwareversion' (with a printer icon), 'Betriebsdaten' (with a refresh icon), 'Auf Gerät hochladen' (with a green upload icon and a mouse cursor), and 'Von Gerät löschen' (with a pill icon). Below the menu, the device information is shown: 'Space plus Perfusor', 'Seriennummer: 16558', 'Softwareversion: 019B0002', and 'Hardwa'. At the bottom, there are two buttons: 'Vom Gerät herunterladen' and 'Auf Gerät hochladen'.

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Below the table, a section titled "Betriebsstatus" shows the update progress. A yellow box highlights the progress bar, which is labeled "Update wird ausgeführt ..." and "Uploading". The progress bar is at 25% completion.

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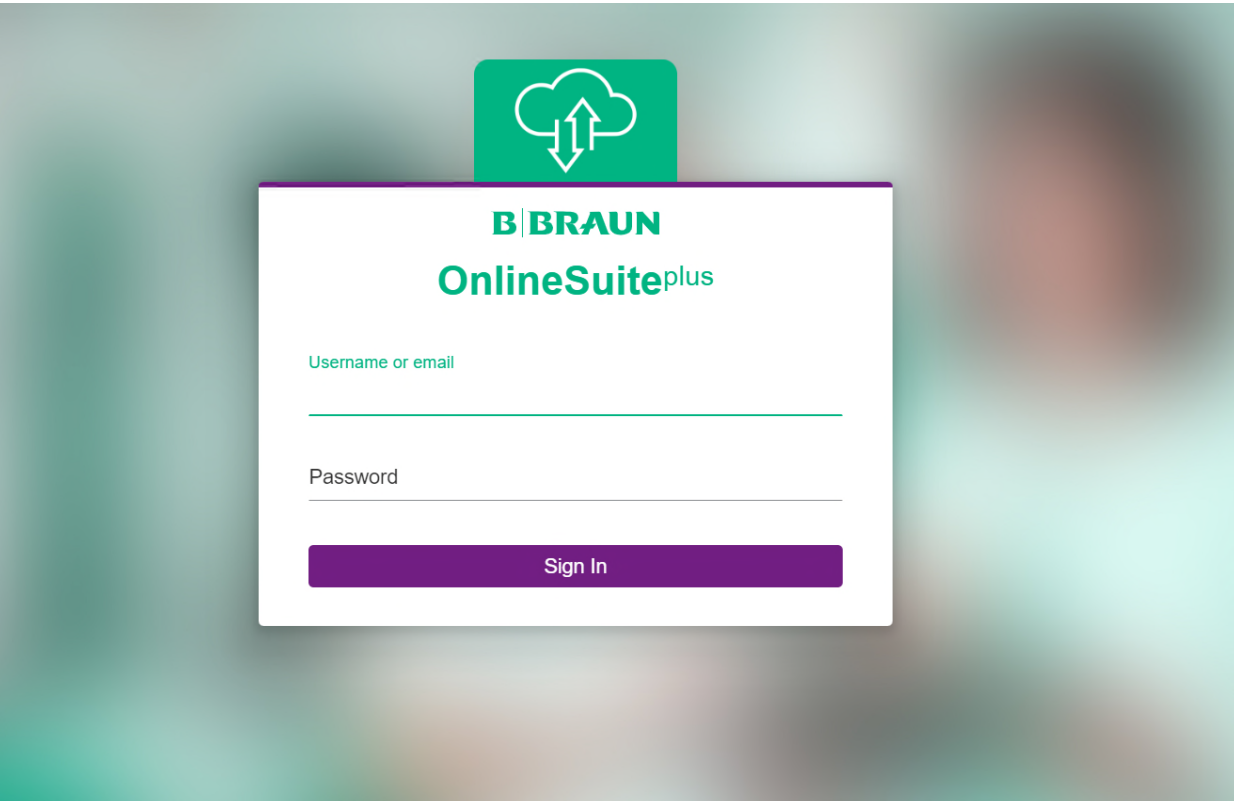
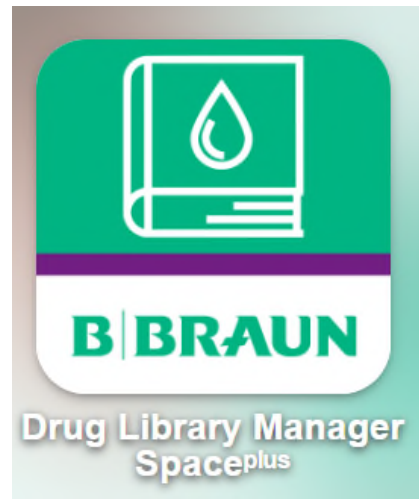
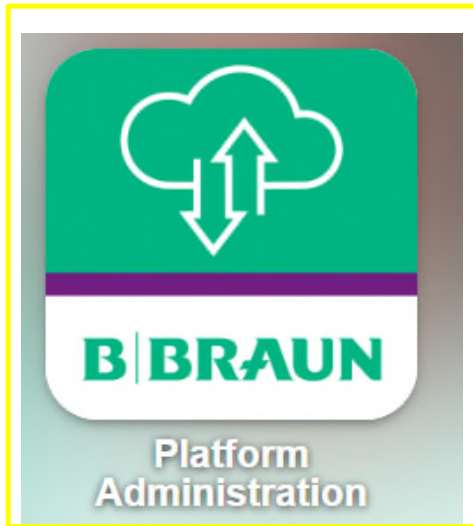
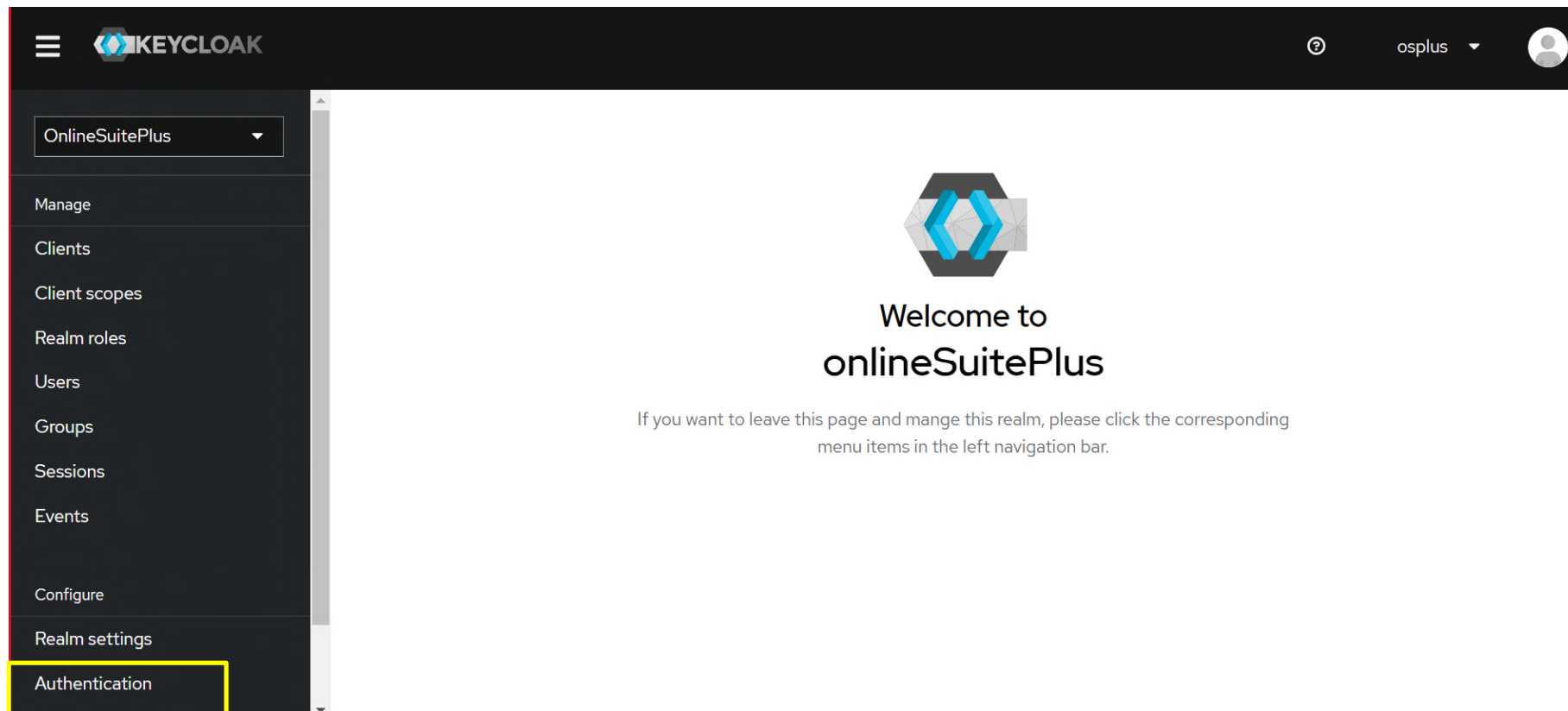
Passwort Einstellungen

The screenshot displays the Keycloak administration interface. At the top left, the Keycloak logo and a hamburger menu icon are present. The top right corner shows a help icon, the realm name 'osplus', and a user profile icon. The left navigation bar is expanded, showing a dropdown menu for 'OnlineSuitePlus' and a list of management options: Manage, Clients, Client scopes, Realm roles, Users, Groups, Sessions, Events, Configure, Realm settings, and Authentication. The 'Authentication' option is highlighted with a yellow border. The main content area features the Keycloak logo, the text 'Welcome to onlineSuitePlus', and a warning message: 'If you want to leave this page and manage this realm, please click the corresponding menu items in the left navigation bar.'

Passwort Einstellungen

The screenshot shows the Keycloak Administration Console interface. The left sidebar contains navigation options: Manage, Clients, Client scopes, Realm roles, Users, Groups, Sessions, Events, Configure, Realm settings, and Authentication. The main content area is titled 'Authentication' and includes a sub-header 'Authentication is the area where you can configure and manage different credential types. [Learn more](#)'. Below this, there are three tabs: 'Flows', 'Required actions' (highlighted with a yellow box), and 'Policies'. A search bar for flows and a 'Create flow' button are also visible. A table lists the following authentication flows:



Flow name	Used by	Description
browser Built-in	✓ Browser flow	browser based authentication
direct grant Built-in	✓ Direct grant flow	OpenID Connect Resource Owner Grant
registration Built-in	✓ Registration flow	registration flow
reset credentials Built-in	✓ Reset credentials flow	Reset credentials for a user if they forgot their password or something
clients Built-in	✓ Client authentication flow	Base authentication for clients
docker authentication Built-in	✓ Docker authentication	Used by Docker clients to authenticate against the IDP

Passwort Einstellungen


The screenshot shows the Keycloak administration interface for the 'OnlineSuitePlus' realm. The 'Authentication' section is active, and the 'Required actions' tab is selected. A table lists various required actions with their status and default settings.

Required actions	Enabled	Set as default action
Configure OTP	<input checked="" type="checkbox"/> On	<input type="checkbox"/> Off
Terms and Conditions	<input type="checkbox"/> Off	<input type="checkbox"/> Disabled off
Update Password	<input type="checkbox"/> Off	<input type="checkbox"/> Disabled off
Update Profile	<input checked="" type="checkbox"/> On	<input type="checkbox"/> Off
Verify Email	<input checked="" type="checkbox"/> On	<input type="checkbox"/> Off

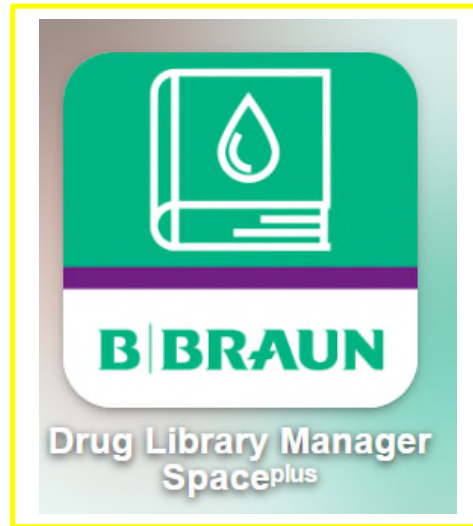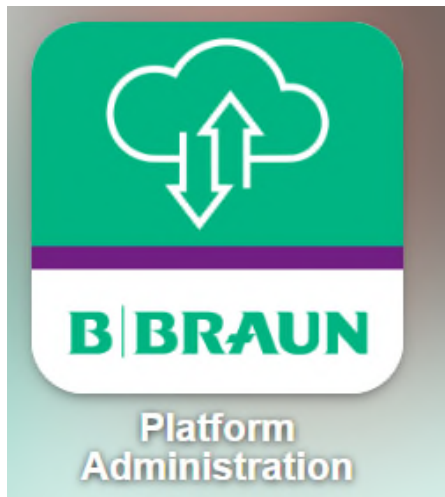Drug Library Manager

Drug Library Manager

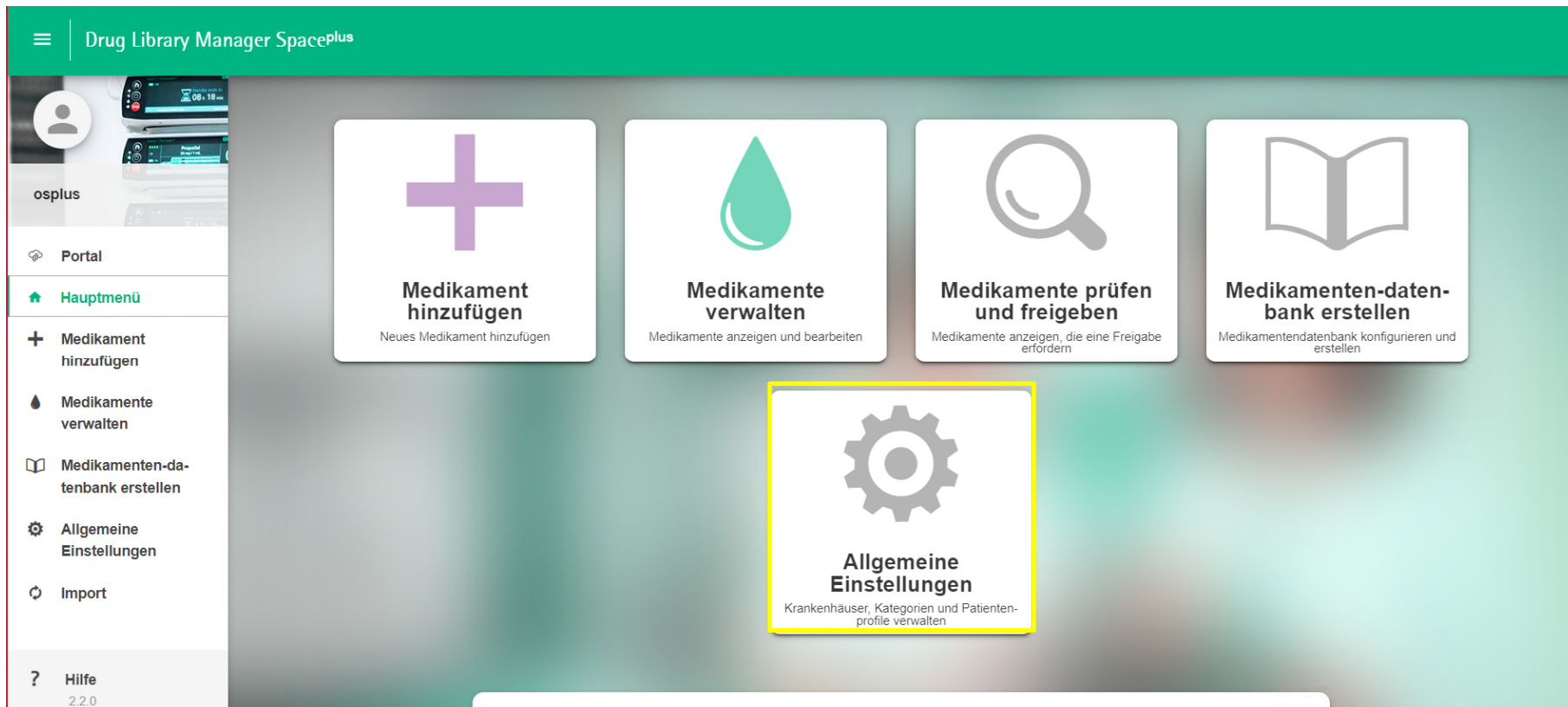Drug Library Manager

The screenshot displays the 'Drug Library Manager Spaceplus' interface. At the top, a green header bar contains a menu icon and the text 'Drug Library Manager Spaceplus'. Below this, a user profile section shows a placeholder icon, the name 'osplus', and a small image of a medical device. The main content area is titled 'Allgemeine Einstellungen' and lists several configuration options: 'Krankenhaus- und Stationseinstellungen' (highlighted with a yellow box), 'Kategorien', 'Patientenprofile', 'Technische Limits', and 'Anwendungseinstellungen'. A left-hand sidebar contains navigation options: 'Portal', 'Hauptmenü', 'Medikament hinzufügen', 'Medikamente verwalten', 'Medikamenten-datenbank erstellen', 'Allgemeine Einstellungen' (highlighted in green), 'Import', and 'Hilfe 2.2.0'.

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At the bottom right, there are two buttons: 'XML-Datei herunterladen' and 'Bearbeiten'.

Drug Library Manager

The screenshot displays the 'Drug Library Manager Spaceplus' interface. The top navigation bar is dark green with a menu icon, the text 'Drug Library Manager Spaceplus', and a 'Gesperrt (von dir selbst)' status indicator. The main content area is titled 'Krankenhaus- und Stationseinstellungen' and features a search bar with the text 'B. Braun Krankenhaus'. A '+ Hinzufügen' button is visible in the top right. A sidebar on the left contains navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten, Medikamenten-datenbank erstellen, Allgemeine Einstellungen (highlighted), Import, and Hilfe 2.2.0. A modal dialog box titled 'Station hinzufügen' is open, showing a dropdown menu for 'Krankenhaus' with 'B. Braun Krankenhaus' selected, and a text input field for 'Stationsname' containing '1. OG'. A 'Speichern' button is at the bottom of the dialog. At the bottom right of the main interface, there are buttons for 'XML-Datei herunterladen' and 'Bearbeiten'.

Drug Library Manager

The screenshot displays the 'Krankenhaus- und Stationseinstellungen' (Hospital and Station Settings) page in the Drug Library Manager SpacePlus. The interface is divided into a left sidebar, a top navigation bar, and a main content area.

- Top Bar:** 'Drug Library Manager SpacePlus' with a menu icon on the left and a '+ Hinzufügen' (Add) button on the right.
- Navigation:** 'Allgemeine Einstellungen > Krankenhaus- und Stationseinstellungen'. A search bar contains '1. OG'. Filter tabs include 'Name', 'Infusion', 'Therapie', 'Alarm', and 'Patient'.
- Sidebar:** Contains navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten, Medikamenten-datenbank erstellen, Allgemeine Einstellungen (highlighted), and Import.
- Main Content:**
 - B. Braun Krankenhaus:** A list of stations with '1. OG' selected and marked 'Unbenutzt'. A '+ Station hinzufügen' button is present.
 - Name:** A table with columns 'Stationsname' and 'Stations-Logo'. The entry for '1. OG' has a logo '1.G'.
 - Infusionseinstellungen:** A section with a warning 'Der Wert kann pro Medikament und pro Therapie neu definiert werden'. It includes:
 - Flussratenlimit:** Two columns for 'Perfusor®' and 'Infusomat®' with descriptions: 'Voreinstellungen der Infusionspumpe anwenden'.
 - Bolusratenlimits:** Two columns for 'Perfusor®' and 'Infusomat®' with descriptions: 'Voreinstellungen der Infusionspumpe anwenden'.
 - VTBI-Vorwahl:** Two columns for 'Perfusor®' and 'Infusomat®'.

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. On the left is a navigation sidebar with the following items: 'Portal', 'Hauptmenü' (highlighted with a mouse cursor), 'Medikament hinzufügen', 'Medikamente verwalten', 'Medikamenten-datenbank erstellen', 'Allgemeine Einstellungen', 'Import', and 'Hilfe'. The main content area features five large, rounded rectangular buttons with icons and text:

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. At the top, a green header contains the title and a menu icon. Below the header, a navigation bar shows four steps: 1. Medikament, 2. Konzentrationen (highlighted), 3. Therapien, and 4. Stationen. The main content area is titled 'Konzentrationen für Propofol 1% (Doseinheit: Gramm) NEU' and includes a lock icon. A 'Medikamenten-Variante' box on the right identifies the drug as 'Basismedikament'. A sidebar on the left lists navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten, Medikamenten-datenbank erstellen, Allgemeine Einstellungen, Import, and Hilfe 2.2.0. The central configuration panel shows '10 mg / 1 mL' with a 'Löschen' button. It includes input fields for 'Wirkstoff' (10), 'Volumen' (1 mL), and 'Einheit' (mg). There are also checkboxes for 'Infusionspumpentyp' (Perfusor® selected, Infusomat® unselected) and 'Konzentrations-ID' (CONT selected, BOT and TGI unselected). A '+ Konzentration hinzufügen' button is in the top right, and a 'Speichern und weiter >' button is highlighted in a yellow box at the bottom right.

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. The top navigation bar is green and contains a menu icon and the text 'Drug Library Manager Spaceplus'. Below this, a breadcrumb trail shows four steps: 1. Medikament, 2. Konzentrationen, 3. Therapien (highlighted in green), and 4. Stationen. The main title is 'Therapien für Propofol 1%' with a 'NEU' badge and a lock icon. A 'Medikamenten-Variante' dropdown is set to 'Basisedikament'. The left sidebar contains navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten, Medikamenten-datenbank erstellen, Allgemeine Einstellungen, Import, and Hilfe 2.2.0. The main content area is divided into sections: 'Therapie-Grundeinstellungen' (Standardtherapie: Nichts., Titration-Guard-Stufe: Aus), 'Aktive Konzentrationen' (10 mg / 1 mL, Standard setzen, Deaktivieren), and 'Inaktive Konzentrationen' (Keine inaktive Konzentration). The 'Kontinuierliche Therapie' section is highlighted with a yellow box and contains sub-sections: 'Infusionsraten-Einstellungen' (Einheit der Infusionsrate: mL/h, highlighted with a yellow box), 'Technische Limits' (Verfügbare Konzentrationen: 1), and 'Infusionsratenlimits' (Obere Limits: Oberes Softlimit, Oberes Hardlimit, Max. 1800). At the bottom right, a purple button 'Speichern und weiter >' is highlighted with a yellow box. The bottom navigation bar includes 'Zurück', ': Weitere Optionen', and the highlighted 'Speichern und weiter >' button.

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. The top navigation bar is green and contains a menu icon and the text 'Drug Library Manager Spaceplus'. Below this, a progress indicator shows four steps: 1. Medikament, 2. Konzentrationen, 3. Therapien, and 4. Stationen. The current step is 'Stationen', which is highlighted in green. The main content area is titled 'Propofol 1% zu Stationen hinzufügen' and includes a 'NEU' badge and a lock icon. A search bar on the right contains the text 'Medikamenten-Variante' and 'Basismedikament'. A toggle switch is labeled 'Automatisch zu allen Krankenhäusern und Stationen hinzufügen'. Below this, a section for 'B. Braun Krankenhaus (1 ausgewählt)' has its own toggle switch 'Automatisch zu allen Stationen hinzufügen'. A list of items is shown, with the first item '1.G 1. OG' highlighted by a yellow box. The bottom navigation bar contains a 'Zurück' button, a 'Weitere Optionen' button, a 'Speichern und Freigeben' button, and a 'Speichern und beenden' button, which is also highlighted by a yellow box. A sidebar on the left lists various navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten, Medikamenten-datenbank erstellen, Allgemeine Einstellungen, Import, and Hilfe 2.2.0.

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. At the top, a green header contains the title and a menu icon. Below the header, a navigation bar includes a user profile icon, a 'Zurück' button, the drug name 'Propofol 1%' with a 'NEU' badge, a '+ Stationsvarianten erstellen' button, and a dropdown menu for 'Medikamenten-Variante' set to 'Basismedikament'. A left sidebar lists navigation options: Portal, Hauptmenü, Medikament hinzufügen, Medikamente verwalten (highlighted), Medikamenten-datenbank erstellen, Allgemeine Einstellungen, Import, and Hilfe 2.2.0. The main content area is titled 'Medikament' and features a sidebar with 'Medikament', 'Konzentrationen', 'Therapien', 'Stationen', and 'Historie'. The 'Medikament' section includes fields for 'Medikamentenname' (Propofol 1%), 'Weitere Medikamentennamen' (-), 'Medikamenten-Info' (-), and 'Kategorien' (-). A 'TCI Medikament' toggle is present. A 'Medikamenten-Etikett Therapiebildschirm' section shows a yellow box with 'Propofol 1%' and a note about a small label in the pump's medication database. An 'Alarmdringlichkeit' (High) setting is also visible. At the bottom, a 'Erweiterte Einstellungen (1 aktiv)' section is partially shown. The bottom right corner contains three buttons: 'Weitere Optionen', 'Bearbeiten', and 'Genehmigen' (highlighted with a yellow border).

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The screenshot displays the 'Drug Library Manager Spaceplus' interface. At the top, a green header contains a menu icon and the text 'Drug Library Manager Spaceplus'. Below this, a navigation bar includes a user profile icon labeled 'osplus', a search bar with 'Suche Medikament' and a magnifying glass icon, and a 'Filtern' button. To the right of the search bar are buttons for 'Stationen', 'Archiv', 'Allgemeine Einstellungen', and a prominent purple '+ Medikament hinzufügen' button. The main content area shows a card for 'Propofol 1%' with a yellow square icon, a 'B.BRN - 1.G' label, and a 'FREIGEgeben' button. A 'Kategorien' field is also visible.

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^ AutoProgramming Medikamentendatenbank

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The screenshot displays the 'Drug Library Manager SpacePlus' interface. At the top, a green header contains the title and a menu icon. Below this, a white header area shows the title 'Neue Medikamentendatenbank' and several input fields: 'Dateiname' (containing 'Demo B Braun' and '_v0.upd'), 'Krankenhaus' (containing 'B. Braun Krankenhaus'), 'Nächste Version' (containing 'Version 1'), and 'Beschreibung'. A left sidebar contains navigation options: 'Portal', 'Hauptmenü', 'Medikament hinzufügen', 'Medikamente verwalten', 'Medikamenten-datenbank erstellen', 'Allgemeine Einstellungen', 'Import', and 'Hilfe'. The main content area is divided into two sections: 'Verfügbare Stationen' and 'Stationen in der Medikamentendatenbank'. In the 'Verfügbare Stationen' section, a card for '1. OG' is highlighted with a yellow box. A 'Hinzufügen' button is also highlighted with a yellow box and has a mouse cursor over it. A '1 Medikament' label is positioned between the two sections. The 'Stationen in der Medikamentendatenbank' section is currently empty, displaying the text 'Fügen Sie eine Station hinzu'. At the bottom right, there are 'Abbrechen' and 'Speichern' buttons.

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