Sorbitol/Mannitol 3 %

B. Braun

bg	Указания за употреба	2
da	Brugsanvisning	3
de	Gebrauchsanweisung	4
en	Information for use	5
es	Instrucciones de uso	6
fi	Käyttöohje	7
fr	Mode d'emploi	8
hr	Upute za uporabu	9
mk	Упатство за употреба	10
nl	Gebruiksaanwijzing	11
no	Brukerveiledning	12
pt	Instruções de uso	13
sv	Bruksanvisning	14

Black

Dimension = 148 x 210 mm
16 pages

Pharmacode: - - -

Irrigation Solution__193_GIF__ZV_0523
193/NU19399/0523
GIF (A5-Booklet, 13-spr.)
Production site: Crissier

Font size: 6,0 pt.

V-0100

B | BRAUN

Sorbitol/Mannitol 3 % B. Braun

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Sorbitol/Mannitol 3%

B. Braun

Irrigation solution, sterile and endotoxin-free

Composition

1000 ml solution contain:

Sorbitol	20 g
Mannitol	10 g
In water for injection	

Indications

- For irrigation during endoscopic examinations of body cavities
- For irrigation during urological interventions by endoscopy involving HF current, e.g. TUR (transurethral resections), disintegration of bladder stones, interventions of the ureter, interventions of the renal pelvis etc.
- For irrigation of gynaecological interventions by endoscopy involving HF current
- For irrigation during arthroscopy of the knee, shoulder or other joints involving HF current
- For postoperative irrigation

Dosage

The volume required will depend on the extent and duration of the intervention and on the postoperative course.

Overdose

During long-lasting interventions, particularly when a large venous sinus is opened, an inflow of irrigation fluid can lead to fluid overload with dilutional hyponatraemia (hypotonic hyperhydration, corresponding to a water intoxication).

For details and symptoms please refer also to the side effect section (TURP syndrome). The symptoms can also appear during the decremental phase of an intervention.

Countermeasures

If this diagnosis is suspected, these symptoms require the intervention to be stopped after careful haemostasis. The natraemia and the haematocrit should be rapidly measured and appropriate treatment for hyperhydration with haemodilution should begin – notably by restricting the hydric flow and administering a suitable diuretic.

Warnings

- Do not use for infusion
- Strictly sterile precautions are to be taken when handling the irrigation solution
- Only use the solution if the seal is undamaged and the solution is clear
- Opened containers must not be saved, quantities left over must be discarded
- Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient
- Store out of reach of children

Sorbitol/Mannitol 3% B. Braun should be applied with care in:

- Cardiopulmonary diseases
- Disturbances of renal function

Disturbances of the acid-base balance and disturbances of the water-electrolyte balance should be corrected before use of Sorbitol/Mannitol 3% B. Braun.

In order to prevent TURP syndrome, the following recommendations should be followed:

- Time of surgery should be reduced to the minimum, especially for those procedures at the increased risk of excessive inflow (normally no excess of 60 minutes)
- Pressure in the operating cavity should be kept at a minimum.
- Ensure adequate haemostasis

- Sufficient monitoring of fluid balance must be ensured
- Throughout the procedure, regular checks must be carried out for excessive inflow of irrigation liquid into the systemic circulation.
- An ongoing monitoring of the symptoms typical for TUR syndrome should be ensured during and after surgical procedure (for detail please refer to section Adverse side effects). The symptoms can also appear during the decremental phase of an interventions.

It is necessary to exclude the possibility of an hereditary fructose intolerance or a fructose-1,6-diphosphatase deficiency before commencement of irrigation. If details of the case history are not available or are not unequivocal a fructose intolerance test must be carried out first. If this is not done there is the possibility of severe complications (including a lethal outcome under certain circumstances). Please refer to section Adverse side effects.

Contraindications

Sorbitol/Mannitol 3% B. Braun should not be applied with:

- Oligo-anuria
- Fructose-sorbitol intolerance
- Hypersensitivity to mannitol

Adverse side effects

Sorbitol/Mannitol 3% B. Braun contains sorbitol. If there is undiagnosed hereditary fructose-sorbitol intolerance the administration of solutions containing sorbitol can lead to nausea, hypoglycaemia, elevation of lactate levels, acidosis and acute liver damage, with possible lethal consequences. Please refer to section Warnings. .)

The most commonly observed side effect is the TURP syndrome (the absorption of fluid with hyponatraemia), caused by an excessive inflow of fluid into the circulatory system. This results in:

- Renal insufficiency (especially where there is existing renal damage)
- Haemolysis
- Clotting disorders caused by inhibition of the clotting factor activity or by lowering the coagulation factor concentration through haemodilution
- Hypothermia

TURP syndrome is characterised by the following symptoms:

- Nausea, vomiting
- Neurological problems: headache, sleepiness, agitation, confusion, which can develop into a potentially fatal coma or convulsions
- Visual disturbances
- Hyponatraemia, hypocalcaemia, sometimes pronounced, reduced osmotic pressure, reduced levels of haematocrit and serum protein
- Bradycardia
- Increased blood volume indicated by dyspnoea, variations in blood pressure or even acute pulmonary or cerebral oedema

All these symptoms are linked to an excessive inflow of irrigation liquid into the systemic circulation and indicate intracellular hyperhydration which can be fatal. They can appear during intervention or in the decremental phase.

Often seen after transurethral resection of the prostate, the described symptoms and signs have been referred to as the TURP syndrome, although they have also been described after other urological or gynaecological surgical procedures.

Expiry date

The solution must not be used after the expiry date given on the label.

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B. Braun Melsungen AG