Prontosan® Debridement Pad

Wound debridement pad

Directions for use

Prontosan® Debridement Pad consists of microfibres that debride and an absorbent backing layer. The device is individually packed and sterilised with ethylene oxide. Prontosan® Debridement Pad is intended for single use only and must not be resterilised. The device is sterile as long as the pack remains unopened and undamaged.

Indications

Prontosan® Debridement Pad is used for rapid, effective and safe debridement. Prontosan® Debridement Pad frees the wound from coatings and dead cell residues (debris) and absorbs excess exudates. Intact tissue is spared.

Prontosan® Debridement Pad produces good results even with scaly and necrotic coatings, if they are subject to prior autolytic treatment.

Composition: polypropylene, polyester

Instructions for use

- 1) Remove the old wound dressing.
- 2) Perform debridement:

- a) The unmarked side of the pad is moistened with Prontosan® wound irrigation solution or, if not available, with saline solution or distilled water. If local guidelines exist, these must be observed.
- b) Applying gentle pressure, wipe the moistened side over the wound and/or adjacent skin in a circular motion. Following debridement, it is advisable to irrigate the wound thoroughly for a second time with Prontosan® wound irrigation solution and to continue with standard treatment.

Precautions

- Do not use Prontosan® Debridement Pad as a wound dressing.
- The device must not be used in case of known intolerance or an allergy to one or more of its constituents.
- Prontosan® Debridement Pad must not be cut to size.

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